


SimpLOFy® SALIVA DNA COLLECTION DEVICE


CATALOG # SIMPL-301

INSTRUCTIONS FOR SALIVA COLLECTION USING THE SIMPLOFY® SALIVA DNA COLLECTION DEVICE


1. Do not eat, drink, smoke, or use oral hygiene products for at least 10 minutes before you start the collection process.
2. Rinse the mouth with water and discard. Please wait at least 5 minutes after this rinse to start the collection procedure.
3. During this time review the detailed instructions on the SimpLOFy® Saliva DNA Collection Device insert below.
4. After the five minutes have passed, open the package containing the SimpLOFy® Saliva DNA Collection Device. Place contents on a clean and dry surface. Follow the illustrated instructions below.

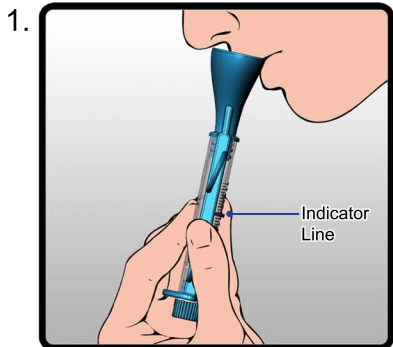


Place lips over the opening of the Collection Tube Funnel. Collect saliva until the sample level reaches the red Indicator Line.

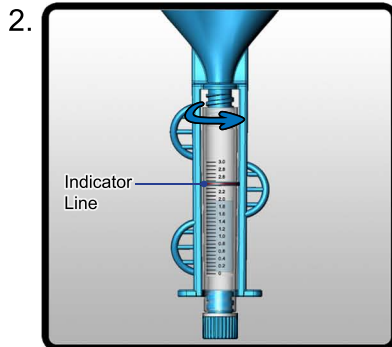

When collection has been completed, unscrew the Funnel from the Collection Tube by rotating the tube in the direction shown until the tube is completely detached from the Funnel.

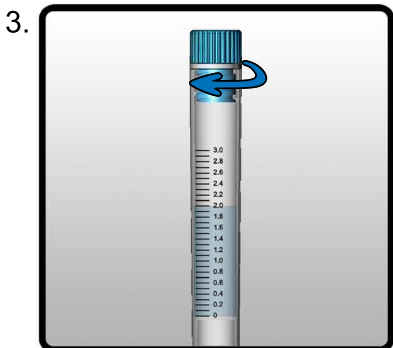

Remove the Cap from the bottom of the Collection Tube and screw it tightly closed.

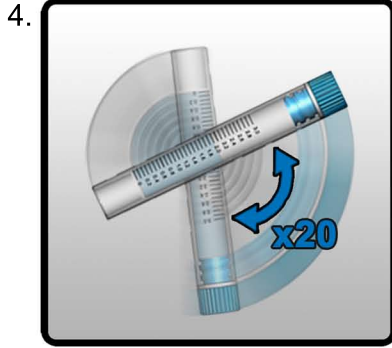

Vigorously invert the vial 20 times to stabilize the sample. Sample is now ready for transportation to a laboratory using standard shipping methods for liquid samples.